


# MEGA MOTION<sup>®</sup>

PRODUCTS FOR **BETTER LIVING**

400 lbs., Infinite Position

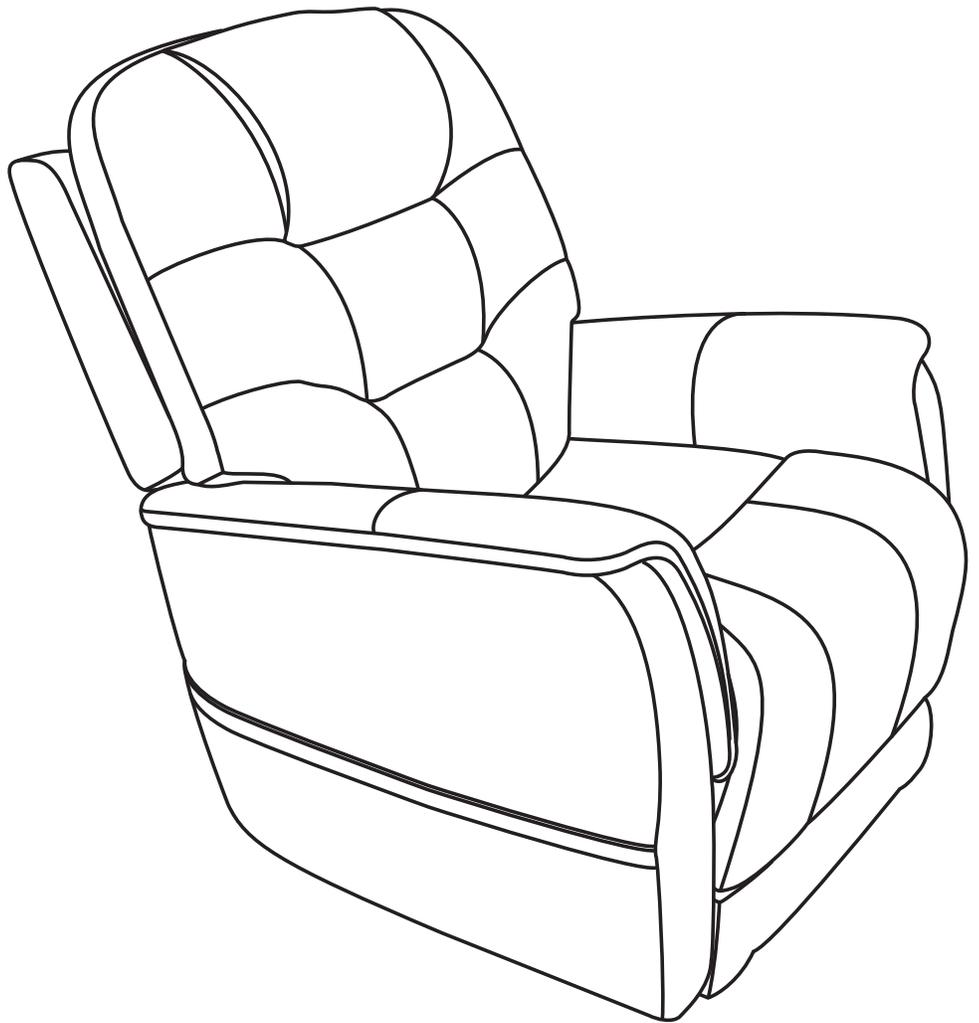

Included in this document: Hand Control Reference Guide, Fabric Care, Chair Assembly and Setup, Warranty, and Weight Capacity



ACN 088 609 661



CH REP

**NOTE: This quick reference guide offers basic hand control functions, general care instructions, basic assembly information, warranty information, and weight capacity information for your chair. Before operating the chair, thoroughly read the owner's manual and all other supplemental documents for information on safety, basic operation, and care and maintenance.**

The illustrations reflect those hand controls with the highest level of functionality. The hand control you receive may have fewer options, but basic functionality is the same.

The premier feature of the Mega Motion Power Recliner is that you have the ability to adjust your feet and back independently to achieve your perfect position. For example, the seat back can recline until it is horizontal to the floor. To achieve this, press and hold button "7" (as shown in **figure 1**) until it is fully reclined.

## HAND CONTROL IDENTIFICATION KEY

### 1. HEAT BUTTON (IF EQUIPPED)

Press the heat button once to activate the gentle heat function(indicated by a green light) press twice to activate the higher heat function(indicated by a orange light) or press three times to activate the highest heat function(indicated by a red light).

### 2. AIR MASSAGE BUTTON (IF EQUIPPED)

Press the air massage button to activate the air massage function. Press the air massage button again to turn off the air massage function.

### 3. HEADREST IN BUTTON

Press and hold to move headrest in.

### 4. HEADREST OUT BUTTON

Press and hold to move headrest out.

### 5. UP BUTTON

Press and hold to return all chair power options to the neutral position, while lifting you into an upright position.

### 6. DOWN BUTTON

Press and hold to lower chair to seated or reclined position.

### 7. SEATBACK RECLINE BUTTON

Press and hold to recline the seatback.

### 8. SEATBACK INCLINE BUTTON


Press and hold to raise the seatback.

### 9. LEG REST UP BUTTON

Press and hold to raise the leg rest.

### 10. LEG REST DOWN BUTTON

Press and hold to lower the leg rest.

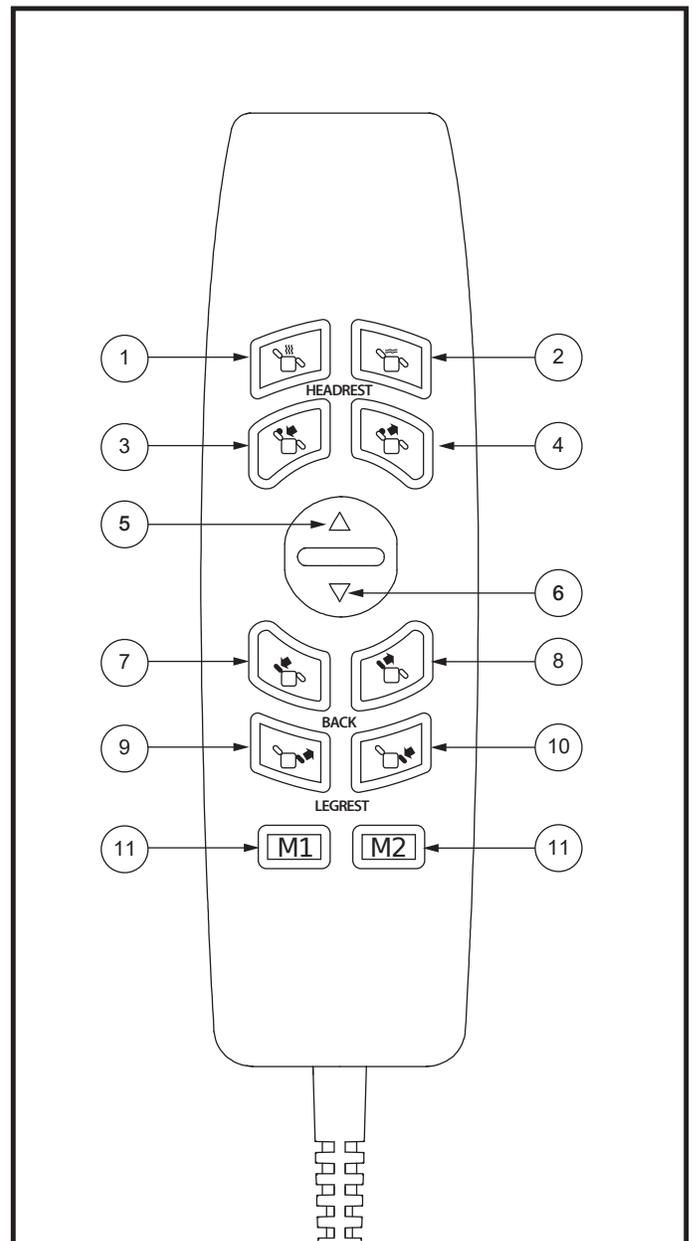

**Figure 1. Hand Control**

## 11. MEMORY BUTTONS

Press and hold Memory Button 1 and Memory Button 2. The LEDs on the memory buttons will flash to indicate that the hand control is in “set” mode. Press and hold the desired memory button to set the memory position. The LEDs will stop flashing. To go to a preset memory position, press and hold the desired memory button.

### Follow these steps to properly stand up from your chair:


1. Activate and hold the UP button (switch) to raise your chair.
2. Release the button (switch) when the chair reaches a height where you can stand up comfortably. If your chair is equipped with cup holders, ensure the cup holders are empty prior to elevating your chair above a seated position to a standing position. **See figure 2.**

**WARNING! Do not attempt to stand up until you can stand safely, are steady on your feet, and can bear weight. Pride recommends the use of assistance aids and/or an attendant for enhanced stability.**



**WARNING! Avoid putting all of your weight on the chair armrests when attempting to stand and do not use the armrests for weight bearing purposes.**

**WARNING! ONLY MFI-certified cables MUST be used when charging iPhone devices equipped with a Lightning Port. Use of any non MFI-certified cable may result in damage to the device or charging system.**

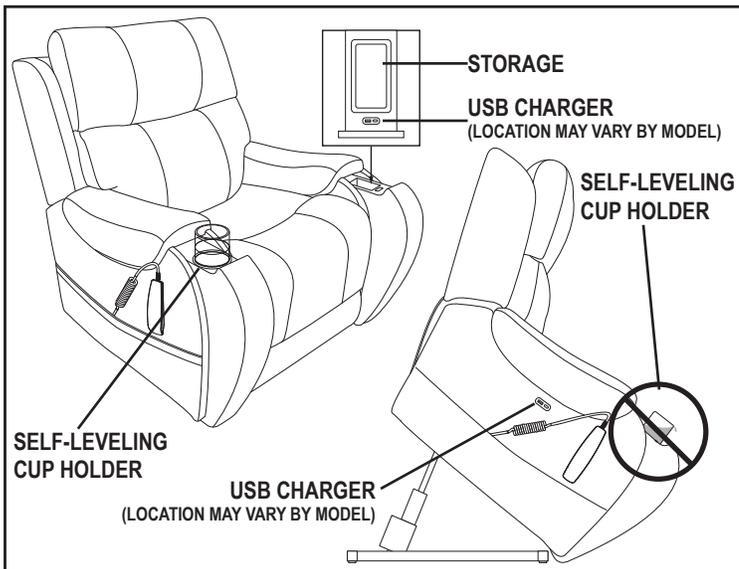


**WARNING! Any contents in the cup holder may shift or spill when the chair is raised. Remove all contents prior to elevating your chair. Failure to remove all contents from the cup holders prior to elevating your chair above a seated position may result in personal injury or property damage.**

**WARNING! NEVER place lit tobacco products in the cup holders/storage or attempt to use the cup holders/storage as an ashtray.**


Figure 2. Self-Leveling Cup Holder, Storage, and USB Charger (IF EQUIPPED)



4. To install the back, align the Knock-Down (KD) sleeves in the back with the KD brackets in the chair base, slide the back down on the base. **See figure 4.** Push down on the top of the back to lock the back into position. You will hear a click from each side of the chair to ensure the back is locked in properly.
5. Plug in the Lumbar and Headrest connectors, **See figure 3.** If your model is Headrest Only, you only need to plug in the Headrest.
6. If your model has back wings, align the KD sleeves, and slide the wing down into place, **See figure 5.** Please note there is a label on the brackets. Please assemble them as indicated (A-A and B-B).
7. Connect the Power Transformer cable to the corresponding plug coming from the back of the chair. **See figure 3.**
8. Connect the Power Cord into the Power Transformer. **See figure 3.**
9. Connect the power cord into an electrical outlet near where the chair will be used.

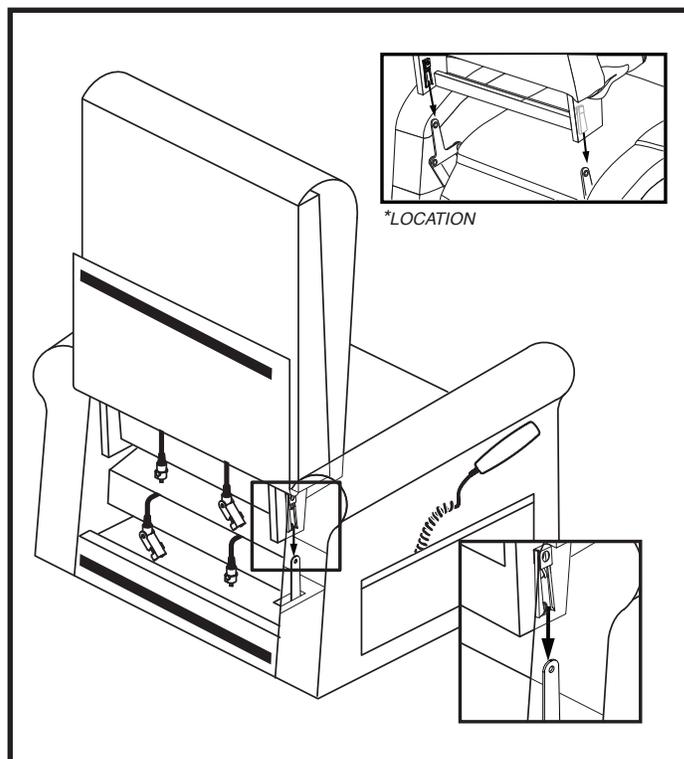

**Figure 4. Chair Back Installation**

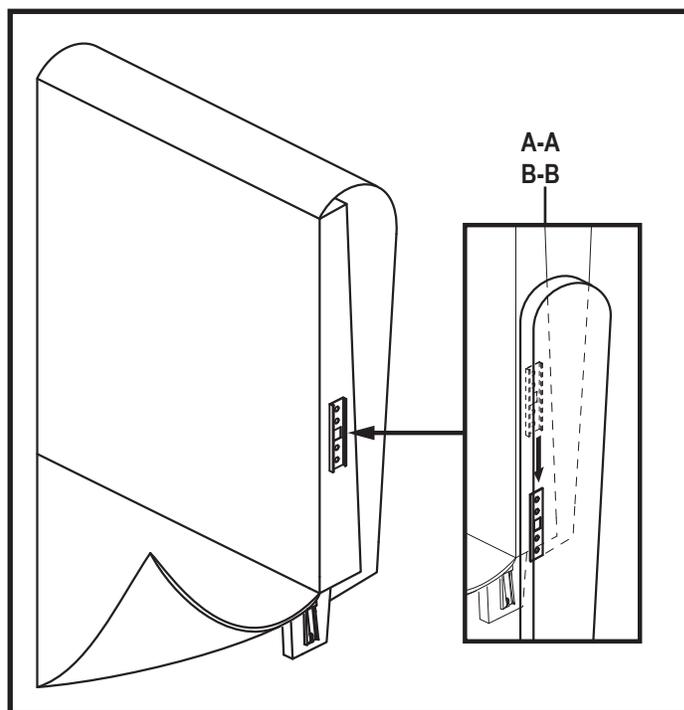

**Figure 5. Wing Back Installation**



### LIFETIME WARRANTY

Mega Motion offers a lifetime warranty on the steel lift mechanism. Mega Motion warrants that the steel lift mechanism on your Power Recliner will be free from defects under normal use and service for as long as the chair is in use by the original owner. This lifetime warranty covers the steel lift mechanism against metal fatigue, defective welds, and bushings.

**NOTE: Mega Motion reserves the right to replace only the part of the steel lift mechanism that may be defective.**

### ONE-YEAR LIMITED WARRANTY

For one (1) year from the date of purchase, Mega Motion will repair or replace at our option to the original purchaser any of the following parts found upon examination by an authorized representative of Mega Motion to be defective in material and/or workmanship:

- Motor
- Hand Control
- Scissor Mechanism
- Transformer/Control Boxes

### ONE-YEAR LIMITED WARRANTY

For one (1) year from the date of purchase, Mega Motion will provide labor, service calls, shipping, and/or any other charges incurred for repair of the product. All transportation costs and shipping damage incurred while submitting parts for repair or replacement are the responsibility of Mega Motion. Please contact your authorized retailer for information on service visits.

This warranty covers service necessitated by product failure during normal usage. The warranty specifically does not cover failure due to customer misuse. The manufacturer reserves the right to determine if the cause of the failure is due to misuse. Customer abuse/misuse will void the warranty.

Service is not extended to fabric, buttons, or wear and tear items or issues. Labor does not cover product maintenance and/or adjustments. Only manufacturing defects are covered.

### RECONDITIONED UNITS WARRANTY

All reconditioned units are covered by a six-month warranty from Mega Motion effective from the date of purchase.

### LITHIUM-ION BATTERY PACK

The lithium-ion battery pack is covered by a twelve (12) month (1 year) warranty.

### WARRANTY EXCLUSIONS

This warranty does not extend to fabric and excludes steel frame chairs.

- Circumstances beyond the control of Mega Motion
- Repairs and/or modifications made to any part without specific consent from Mega Motion

### Exclusions also include components with damage caused by:

- Contamination
- Abuse, misuse, accident, or negligence
- Battery fluid spillage or leakage
- Commercial use or use other than normal
- Improper operation, maintenance, or storage

### IMPLIED WARRANTIES

**Implied warranties, including those of merchantability and fitness for a particular purpose, are limited to one (1) year from the date of purchase and to the extent permitted by law. Any and all other implied warranties are excluded.** This is the exclusive remedy. Liabilities for consequential damages under any and all warranties are excluded.

Some states do not allow limitations on how long an implied warranty lasts or do not allow the exclusion or limitation of incidental or consequential damages. The above limitation or exclusion may not apply to you.

This warranty gives you specific rights, and you may also have other rights which vary from state to state.

## WEIGHT CAPACITY



Your chair was developed for your convenience. Your chair has a weight capacity of 400 lbs. (181 kg). Refer to the owner's manual for any additional information concerning your chair.



**MANDATORY!** Stay within the specified weight capacity of your Power Recliner. Exceeding the weight capacity voids your Power Recliner warranty.

Rev A/January 2026



I N F I N I T Y B 4 8 7 4